

**AMS INTERNATIONAL**

**The Old Vicarage  
Tuxford  
Newark  
Notts  
NG22 0HP**

## **Toothbrushing Programmes**

### **S-RACKS & LIDS**

#### **Cleaning Instructions**

**02/09/2020**

- Rinse with hot water
- Clean in warm/hot soapy water using standard detergents (maximum temperature = 75°C)  
or cold sterilize
- Dishwash on top shelf only
- Do not steam, boil or microwave
- Air-dry or dry with clean paper towels

**Before first use wash thoroughly.  
Clean and dry weekly (more if soiled)**

**S-Racks and lids should be replaced if cracks, scratches or rough surfaces develop.**